
Tip for bone cutting

VarioSurg Tip

OPERATION MANUAL

Please read this OPERATION MANUAL
carefully before use, and file for future
reference.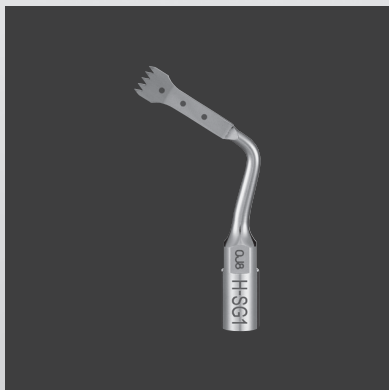

Thank you for purchasing this product.

This tip is for cutting the jawbone. Before use, please be sure to carefully read this operation manual and keep it accessible for future reference.

1 User and Indications for Use

User: Qualified Professionals

Indications for Use: The device is intended for bone cutting in oral surgery.

2 Precautions for Handling and Operation

- Please read these precautions carefully and use only as intended or instructed.
- Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device. Safety instructions are classified as follows in accordance with the seriousness of the risk.

Class	Degree of Risk
 WARNING	Hazard that could result in serious injury or damage to the device if the safety instructions are not correctly followed.
 CAUTION	Hazard that could result in light or moderate injury or damage to the device if the safety instructions are not correctly followed.
NOTICE	General product specification information highlighted to avoid product malfunction and performance reduction.

WARNING

- Be sure to always use with sufficient coolant water. Insufficient coolant water could cause a burn of the affected area or the overheating of the instrument.
- Be aware of the tip end position. Constantly move the tip so that it does not stagnate at one point. If the tip is stagnated at a single point, there will not be enough cooling, and this could result in overheating of the bone or excessive cutting of the bone.
- Do not bring the tip into contact with soft tissue such as gums or mucous membranes for an extended period of time. This could result in a burn injury.


CAUTION

- This tip is specially designed for the Ultrasonic Bone Surgery System "VarioSurg series" by NSK. Do not use with other systems.

 CAUTION


- This product must be used by a qualified personnel in accordance with the indications for use and the directions for use stated in this operation manual.
- Set the output based on "4-2 Output Settings".
- Without coolant water the temperature of the Tip may reach 65°C. To avoid this event, be sure to use coolant water.
- Do not allow any impact on to the product. Do not drop the product.
- Operators and all others in the area must wear eye protection and a mask when operating this tip.
- Prior to clinical use, inspect the tip. Check for vibration, noise and overheating. If any abnormalities are found, stop using the tip immediately and contact your Authorized NSK Dealer.
- Before use, be sure to always check that the output is set to the proper settings.
- Before use, check that the tip is firmly attached to the handpiece.
- The tip is a wear item. Replace with a new tip as early as possible based on the timetable provided in "6 Tip Replacement Interval". Also, the tip can break due to metal fatigue. Replace with a new tip on a regular basis.
- During vibration, do not grasp the tip nor make contact with the tip for an extended time.
- Do not grind or bend the tip. This could break the tip or cause it to fail to vibrate.
- Do not use tips that are damaged, bent, corroded, or show other imperfections. The tip could break or fail to vibrate.
- Do not apply more pressure than required to the tip end. This could cause the tip to break or warp.
- Do not twist or strain the tip, and do not use it as a replacement for surgical tools. This could cause the tip to deform or break.
- If the tip cracks or breaks during use, immediately stop operation, and replace with a new tip. Operation with a damaged tip could injure the gums, periodontal tissue, or other locations.
- The coating on the tip surface (TiN coating) may peel off depending on the number of applications (typically around 5 applications) and the bone condition. If the coating peels off, the smooth glide of the tip surface becomes less effective, resulting in reduced cutting performance. If you sense this type of condition, replace with a new tip.
- Do not use on patients who use a cardiac pacemaker.
- The tip is delivered in a non-sterile condition and must be autoclaved prior to use.
- U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

3 Model and Order Code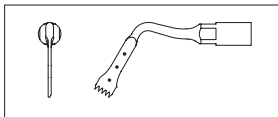


Model: SG1

Order Code: Z305101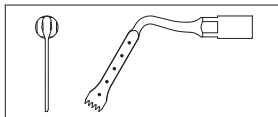


Model: SG1A

Order Code: Z305138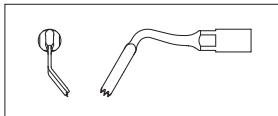


Model: SG2R

Order Code: Z305102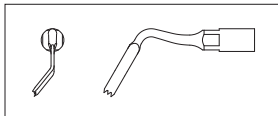


Model: SG2L

Order Code: Z305103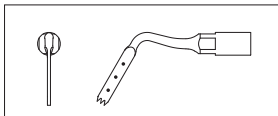


Model: SG8

Order Code: Z305109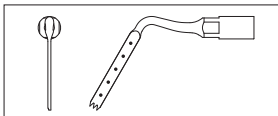


Model: SG8A

Order Code: Z305139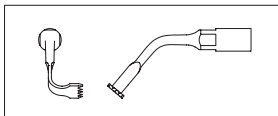


Model: SG14R

Order Code: Z305122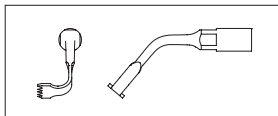


Model: SG14L

Order Code: Z305123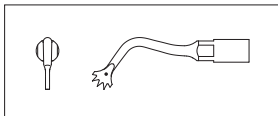


Model: SG19

Order Code: Z305135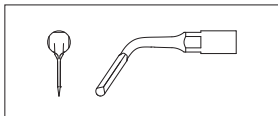


Model: SG30


Order Code: Z305137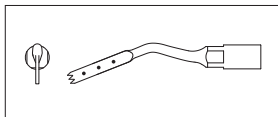


Model: SG58
Order Code: Z305141

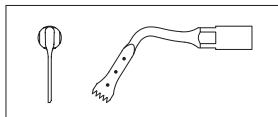
Model: H-SG1
Order Code: Z305151

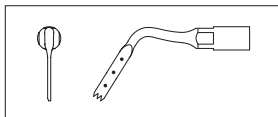Model: H-SG8
Order Code: Z305155

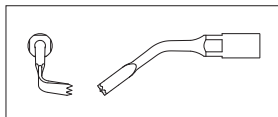Model: H-SG8R
Order Code: Z305156

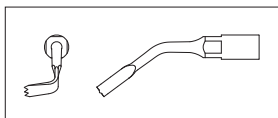Model: H-SG8L
Order Code: Z305157


Example of marking

H-SG1

Marking
This is a guide for
the cutting depth.

H-SG8L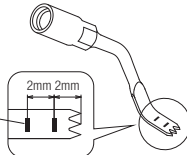

Tips with "•" markings (3mm intervals): SG1, SG1A, SG8, SG8A, SG19, SG58, H-SG1, H-SG8

Tips with "-" markings (2mm intervals): H-SG8R, H-SG8L

Tips without marking: SG2R, SG2L, SG14R, SG14L, SG30

4 Directions for Use

4-1 Attachment

- 1) Screw the tip into the handpiece by hand until it is tightened lightly (Fig. 1).
 - 2) Locate the tip wrench aligning with the square portion of the tip (Fig. 2).
- Next, tighten the tip wrench (clockwise) until it turns freely and clicks (Fig. 3). At that time, hold the handpiece so as not to kink the handpiece cord. To remove the tip, use the tip wrench and turn (anticlockwise) in the "Loosen" direction in Fig. 3.

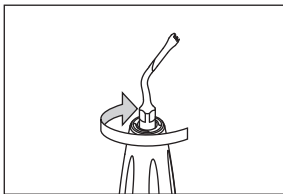
Fig. 1

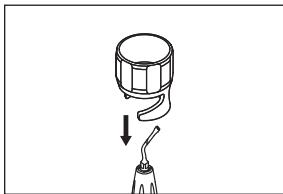
Fig. 2

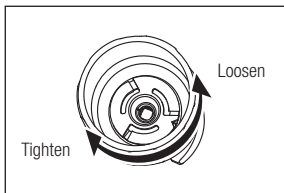
Fig. 3

Tip Usage Caution

- If you attach a tip with debris adhering to the threaded section, vibrations may become weaker, or no vibrations may be generated. Be sure to clean and remove debris from the threaded section.
- If the tip vibration is not observed during use, remove the handpiece from the patient's mouth, then restart the handpiece operation. If this does not improve the situation, the tip may have loosened during use. Check that the tip is securely attached.
- When attaching the tip, always wear sterilized gloves, and use a tip, handpiece, and tip replacement wrench that are all sterilized.
- The tip replacement wrench is a wear item. Replace it once a year.
- Securely tighten the tip using the tip replacement wrench. If the tightening force is too weak, the vibrations may become weaker, or no vibrations may be generated.

4-2 Output Settings

- Do not use the tip at a setting higher than the Max. power level written on the tip case.
- Always apply the tip from the lowest power level and adjust the power while observing the patient condition.

CAUTION

- Be sure to always use the tip at a setting not exceeding the Max. power level. If the tip is used in excess of the maximum operating power, the tip could break, the bone could overheat, or excess cutting could occur.
- If using at a high power setting or in an overload state, for certain tips, the handpiece surface can start to feel warm (the surface temperature can exceed 41°C while supplying coolant water). If this happens, stop using the handpiece for about 5 minutes, and wait until the surface cools before using again.

* Be sure to always carefully read the operation manual for the VarioSurg series before use.

4-3 Usage Procedure

- 1) Attach the tip. (See "4-1 Attachment".)
- 2) Set the output for the VarioSurg series ultrasonic system so that it does not exceed the Max. power level written on the tip case. (See "4-2 Output Settings".)
- 3) Adjust the water flow volume and direction of the tip to the cutting section so that the coolant water is sprayed to the tip end.
 - * For details on operation of the VarioSurg series, see the specific product operation manual.
- 4) Perform bone cutting for the specific application.

5 Post-use Maintenance

After each patient maintain the product as follows.

5-1 Cleaning

Remove debris on the tip surface using a brush (Do NOT use a wire brush). Then wipe the surface with alcohol-immersed cotton swab or cloth.

5-2 Sterilization

After each patient sterilize as noted below.

- 1) Place the tip onto the tip holder, then place it in the sterilization cassette.
- 2) Autoclavable under the conditions below.
 - Autoclave for more than 20 min. at 121°C, or 15 min. at 132°C, or 3 min. at 134°C.
- 3) Keep the tip in the sterilization cassette until required for use.

CAUTION

- Do not autoclave the product with other instruments even when it is in a pouch. This is to prevent possible discoloration and damage to the product from chemical residue on other instruments.
- Do not heat or cool the product too quickly. Rapid change in temperature could cause damage to the product.
- To avoid product failure, do not use a sterilizer that exceeds a cycle temperature of 138°C, including the dry cycle. In some sterilizers, the chamber temperature may exceed 138°C. Contact the sterilizer manufacturer for detailed information about cycle temperatures.
- Keep the product in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. The air should be free from dust, salt and sulphur.
- Do not touch the product immediately after autoclaving as it will be very hot and must remain in a sterile condition.
- Autoclave sterilization is recommended for the product. The validity of other sterilization methods is not confirmed.
- Do not use electrolyzed-oxidizing water (strong acidic water or super acidic water) or sterile liquid for washing, immersing, or wiping.
- Before storing after use, be sure to thoroughly dry off any remaining moisture. Storing while moisture remains can result in damage due to rust and other adverse effects.

NOTICE ● NSK recommends Class B sterilizers as stated in EN 13060.

6 Tip Replacement Interval

Although it depends on the bone condition and usage time, the tips can generally be used for around 5 treatment applications.

Always replace the tip whenever it shows signs of wear or becomes deformed.

7 Symbol

This product is Autoclavable up to Max. 135°C.

Conforms to CE European Directive of "Medical device directive 93/42/EEC."

Manufacturer

Authorized representative in the European community.

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed physician.

8 Disposing of the Product

In order to avoid the health risks of operators handling the disposal of medical equipment, as well as the risks of environmental contamination caused thereof, a surgeon or a dentist is required to confirm the equipment is sterile. Ask specialist firms who are licensed to dispose of specially controlled industrial wastes, to dispose the product for you.

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NSK Europe GmbH
NSK France SAS
NSK United Kingdom Ltd
NSK America Corp
NSK Oceania Pty Ltd